

## APPROVED CLEANING PRODUCTS FOR MATTRESS COVERS, UPHOLSTERY TOPS AND SHOWER TROLLEY TUB FABRICS:

APPROVED CLEANING PRODUCTS	MAXIMUM CONCENTRATION
Actichlor Plus	Use according to the manufacturers specifications.
Actimax	Use according to the manufacturers specifications.
ALCO SANNI	Use according to the manufacturers specifications.
ALCO-DC-7	Use according to the manufacturers specifications.
Alcohol wipes 70% Isopropyl Alcohol	Use according to the manufacturers specifications.
Arjo Disinfectant	Use according to the manufacturers specifications.
Azo max multi surface wipes	Use according to the manufacturers specifications.
Azo wipe hard surface wipes	Use according to the manufacturers specifications.
Chlor-Clean Detergent	Use according to the manufacturers specifications.
Chloradet	Use according to the manufacturers specifications.
Cidal, disinfectant/cleaner	Use according to the manufacturers specifications.
Clinell Detergent Wipes	Use according to the manufacturers specifications.
Clinell Hand & Surface Sanitizer Disinfectant	
Clinell Sporidical Wipes (GlobalMedics)	
Clinell Universal Sanitising Wipes	
Clinimax Difficil - S	Use according to the manufacturers specifications.
Dettol	Use according to the manufacturers specifications.
Ecopine	5% Ecopine (1 part Ecopine to 20 parts water)
Hygena wash, shampoo and body wash	Use according to the manufacturers specifications.
JUMBO Liv-Wipes	Use according to the manufacturers specifications.
Leggeside HB (Legge Systems)	Use according to the manufacturers specifications.
Mediwipes Disinfection (Sulco Limited)	Use according to the manufacturers specifications.
Mikrozyd® universal liquid	Use according to the manufacturers specifications.
Nocospray - hydrogen peroxide	Use according to the manufacturers specifications.
NON ACID bathroom cleaner	Use according to the manufacturers specifications.
PAL Detergent Wipes	Use according to the manufacturers specifications.
PAL Disinfectant Wipes (GlobalMedics)	
PCS 5000 wipes	Use according to the manufacturers specifications.
Perform® sterile PAA	Use according to the manufacturers specifications.
Reynard Health Supplies	
Neutral Detergent Wipes	Use according to the manufacturers specifications.
Surface Disinfecting Wipes	Use according to the manufacturers specifications.
Premier Disinfecting Wipes	Use according to the manufacturers specifications.
PureGreen 24 (Pure Green)	Use according to the manufacturers specifications.
Sanicare TBX	Use according to the manufacturers specifications.
Sanicare Disinfectant Wipes (Buckeye International)	
Sani-Cloth	Use according to the manufacturers specifications.
Sporeclear	Use according to the manufacturers specifications.
Spring Clean (Eco Lab)	5% Spring Clean (1 part Spring Clean to 20 parts of water)
Sterri-Matt Detergent	Use according to the manufacturers specifications.
Stride Neutral Detergent	Use according to the manufacturers specifications.
Super Sani Cloth -Germicidal Disposable Wipes (PDI)	Use according to the manufacturers specifications.
S-7 Xtra Wipes	Use according to the manufacturers specifications.
Terminator One Step Disinfectant (Buckeye International)	Use according to the manufacturers specifications.
Therasol	Use according to the manufacturers specifications.
TPH protect	Use according to the manufacturers specifications.
Triad (by Diversey)	Use according to the manufacturers specifications.
Trigene (Bio Strategy)	Use according to the manufacturers specifications.
Tristel Fuse	Use according to the manufacturers specifications.
Tuffie Wipes	Use according to the manufacturers specifications.



---

Tuffie 5 Cleaning Wipes (Vernacare)	Use according to the manufacturers specifications.
Turbo cleaner, hard surface cleaner/detergent	Use according to the manufacturers specifications.
Viracleans Benzalkonoim Chloride	Use according to the manufacturers specifications.
Virkon (Dupont)	5% Virkon (1 part Virkon to 20 parts water)

---

**Example - Use 5% Virkon (1 part Virkon to 20 parts of water)**

Use 25ml per 0.5 litres of water. Use 50ml per 1 litre of water. Use 100ml per 2 litres of water. Use 150ml per 3 litres of water.

## CLEANING INSTRUCTIONS

### GENERAL CLEANING INSTRUCTIONS (ALL SURFACES)

**To wash:**

- Wipe with a sponge or soft cloth wetted with warm water containing soap or mild detergent.
- The mattress cover can be machine washed (95°C max).

**To dry:**

- Wipe with a dry sponge or soft cloth.
- Place mattress cover in a dry environment (if further drying is required).
- Cover can be tumble dried at a temperature below 50°C but must be removed from the drier immediately when dry and not be left in the machine. Interrupting the drying cycle occasionally to redistribute the covers may reduce drying time (only bielastic mattress covers).

**To disinfect:**

- Wash the surface.
- Wipe surface with a sponge or soft cloth wetted with an approved cleaning product.
- Wipe off residue with water using a soft cloth or sponge.
- Dry.

**Steam Cleaning (with or without cleaning chemicals):**

- Steam cleaning can be performed if necessary but is not recommended for regular use.
- Steam cleaning should be performed by carefully following the manufacturer's instructions.
- During steam cleaning, castors, electric components and mattresses should be avoided.
- Any remaining residue should be wiped dry. If cleaning chemicals are used, these should be rinsed off with clean water before wiping dry.

### BLEACH CLEANING INSTRUCTIONS

If none of the approved cleaning products provide the disinfectant level required then use a solution of Sodium Hypochlorite diluted to 1000ppm.

Bleach products are corrosive and degrading in nature and may cause damage to your product. If bleach is used, measures must be taken to ensure the product is rinsed thoroughly with clean water and thoroughly dried. Failure to rinse and dry the product will leave a corrosive residue on the surface possibly causing corrosion.

Failure to follow the above directions may void the product's warranty.