

PREMA MATTRESS RANGE CLEANING INSTRUCTIONS

CAUTION

Before cleaning:

- Remove all linen from mattress

During cleaning:

- Do not use abrasive cleaning products
- Do not use a water temperature of more than 50°C
- Do not use a steam cleaner, washing tunnel, high-pressure spray or hose
- Do not immerse the product in water
- Use only approved cleaning products. If bleach is necessary strictly adhere to the BLEACH CLEANING instructions
- Do not use oil based products
- Keep sharp objects such as razors etc away from the mattress cover

ALL SURFACES

To wash:

- Wipe with a sponge or soft cloth wetted with warm water containing soap or mild detergent
- Cover can be machine washed in warm water (only bielastic mattress covers)

To dry:

- Thoroughly wipe with a dry sponge or soft cloth
- Place mattress cover in a dry environment (if further drying is required)
- Cover can be tumble dried at a temperature below 50°C but must be removed from the drier immediately when dry and not be left in the bowl. Interrupting the drying cycle occasionally to redistribute the covers may reduce drying time (only bielastic mattress covers)

To disinfect:

- Wash the surface with one of the approved cleaners or refer to the Bleach cleaning instructions if bleach needs to be used
- Wipe off residue with water using a soft cloth or sponge
- Thoroughly dry

REMEMBER

If a washing machine is used, interrupt the washing cycle occasionally to redistribute the covers. This will ensure that they are properly wetted.

Dry cleaning is not recommended.

MATTRESS FOAM

The PREMA mattress cover is waterproof and vapor permeable and it features welded seams and a waterproof zip. If the mattress cover is punctured it should be replaced to avoid contamination of the mattress foam.

Excluding the PREMA Stretcher mattress, it is recommended that the mattress is periodically flipped over and rotated.

BLEACH CLEANING INSTRUCTIONS

If none of the approved cleaning products provide the disinfectant level required then use a solution of Sodium Hypochlorite diluted to 1000ppm.

Bleach products are corrosive and degrading in nature and may cause damage to your product. If bleach is used, measures must be taken to ensure the product is rinsed thoroughly with clean water and thoroughly dried. Failure to rinse and dry the product will leave a corrosive residue on the surface possibly causing corrosion.

Failure to follow the above directions may void the product's warranty.

APPROVED CLEANING PRODUCTS

Please refer to www.howardwrightcares.co.uk for approved cleaning products and maximum concentration.