

RESET INSTRUCTIONS

All products have to be plugged into an electrical power supply before doing the reset.

M9 STRETCHER

FOR STRETCHERS BUILT BEFORE AUGUST 2015

To reset control box:

Using the handset, press and hold both the DECK UP and DECK DOWN buttons simultaneously. The control box will beep 10 times. When the beeping stops, the control box is reset.

To initialise control box:

After a reset, using the handset, you must raise the deck to full height.

FOR STRETCHERS BUILT AFTER AUGUST 2015

To reset control box:

Using the handset, press and hold both the DECK UP and DECK DOWN buttons simultaneously. The control box will beep 10 times. When the beeping stops, the control box is reset. After a reset you must initialise the control box.

To initialise control box:

Using the handset, press and hold down the top two buttons (BACKREST UP and BACKREST DOWN). When the stretcher stops moving and the beeping stops, the control box is initialised.


M9 WARD AND COMPACT BED

To reset control box:

Using the standard handset (see image on the right), press and hold both the DECK UP and DECK DOWN buttons simultaneously. The control box will beep 10 times. When the beeping stops, the control box is reset.

To initialise control box:

After a reset, using the standard handset, you must raise the deck to full height.

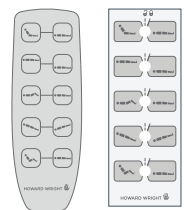


M8 INTENSIVE CARE BED

To reset control box:

Using the standard handset (see image on the right), press and hold both LEGRAISE UP and LEGRAISE DOWN buttons simultaneously. Bed will beep several times. When beeping stops control box is reset.

To initialise control box:

Press each function on the standard handset until each actuator reaches its full extension to initialise.

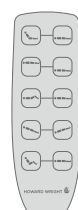

M10 WARD AND COMPACT BED

To reset control box:

Using the handset, press and hold both the DECK UP and DECK DOWN buttons simultaneously.

The control box will beep 10 times. When the beeping stops, the control box is reset. After a reset you must initialise the control box.



To initialise control box:

Using the handset, press and hold down the top two buttons (BACKREST UP and BACKREST DOWN).

When the bed stops moving and the beeping stops, the control box is initialised.